


1

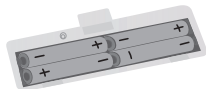

Insert batteries.

2



Place on hard, flat surface.
Step on the scale.

3



Step off the scale.

4



Measurements sent to your
healthcare provider.


Note: Once the batteries are inserted, it may take several minutes for the scale to connect with the cellular network to set the clock. The length of time will depend on the network strength in your area. Do not power off.

TIPS FOR TAKING WEIGHT



- Ensure scale is located in area with good cellular signal.
- Place on hard, flat surface.
- Try to measure at the same time everyday, like when you wake up and after you go to the bathroom.
- Remain still to obtain accurate measurement.

1

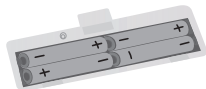

Coloque las pilas suministradas en el bascula.

2



Colocar sobre una superficie dura y plana. Párese en el bascula para pesarse.

3



Bájese de el bascula.

4



Medidas enviadas a su proveedor de atención médica.

Nota: Una vez que se insertan las baterías, la báscula puede tardar varios minutos en conectarse con la red celular para configurar el reloj. El período de tiempo dependerá de la potencia de la red en su área. No lo apague.

CONSEJOS PARA TOMAR PESO



- Asegúrese de que la báscula esté ubicada en un área con buena señal celular.
- Colóquelo sobre una superficie dura y plana.
- Intente medir a la misma hora todos los días, como cuando te despiertas después de ir al baño.
- Permanezca quieto para obtener una medición precisa.